Suprasorb® P sensitive border lite, multisite border lite

Silikon-Schaumverband
Pansement hydrocellulaire siliconé
Silicone Foam Dressing
Apósito de espuma y silicona
Ligadura esponjosa de silicone
Medicazione in schiuma di silicone
Siliconen schuimverband
Silikone-skumforbinding
Silikonskumförband
Silikon-skumbandasje
Silikonivaahotosidos
Silikonový pěnový obvaz
Silikónový penový obväz
Szlikon habkötés
Opatrunek z pianki z warstwą silikonu
Obloga iz silikonske pene
Povoj sa silikonskom pjenoj
Απώστρες σιλικονοϋ επιθέματα
Губчачаста пов'язка з сіліконом
Silikonlu köpük yara örtüsü
ضمادة السيليكون الإسفنجية

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Instructions for use

Product description

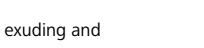
Suprasorb® P sensitive border lite consists of a silicone wound contact layer, an absorbent polyurethane foam, a non-woven distributing layer and a protective vapour permeable and waterproof backing. The four-layer construction facilitates dynamic fluid management to provide and maintain optimal moist wound environment which leads to the promotion of faster wound closure and may help to reduce the risk of maceration. The gentle silicone layer can be lifted and repositioned without losing its adherence. Also, this layer can minimise pain to patients and trauma to wounds and the surrounding skin during dressing changes. The PU foam layer provides superior absorption and locks exudates away.

Product composition

Silicone, polyurethane, polyacrylate, viscose and polyester

Intended purpose

Suprasorb® P sensitive Silicone Foam Dressing may be used to treat all wounds, provided they are

lightly to moderately exuding and

superficial to

deep*.

Indications

- Pressure ulcers
- Leg and foot ulcers
- Traumatic wounds
- Surgical wounds
- Skin tears

Contraindications

Known allergy and/or hypersensitivity to any of the product components

Application instructions

May be applied by lay persons under supervision of health care professionals.

The product can be used under compression bandaging.

- Preparing the wound
 - Clean the wound in accordance with current clinical standards. Dry the surrounding skin thoroughly.
- Applying the wound dressing
 - The size of the dressing depends on the wound size. The wound pad should cover the dry surrounding skin by at least 1-2 cm.

- Remove the first release liner and apply the adherent side to the wound.
- Remove the remaining release liner(s) and smooth down the border on the skin. Do not stretch the dressing.
 - *Where necessary, for deep wounds apply a rope as primary dressing (e.g. Suprasorb® A, Suprasorb® A+Ag, Suprasorb® Liquecel).
- Dressing change
 - The dressing can remain on the wound for several days, dependent on wound condition and the level of exudate.

Precautions and warnings

- The product is for single use only.
- Do not use it if the packaging is damaged or opened.
- Do not use Suprasorb® P sensitive Silicone Foam Dressing together with oxidising agents such as hypochlorite solutions or hydrogen peroxide.
- If reddening or sensitisation occur discontinue use and consult a health care professional for appropriate treatment.
- Because of the lack of data, cautious use is advised in sensitive population groups such as infants, children, pregnant or nursing women.

Note

In case of serious incident, contact the manufacturer and the responsible health authorities.

Storage and disposal

- Keep in a clean and dry place in specified temperature range and away from sunlight.
- Disposal should be handled according to local environmental procedures.

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