Physiodose

FR Sérum physiologique stérile

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EN Sterile Saline Solution

Physiodose saline solution is a sterile, preservative-free, non-injectable, saline solution. Presented in single-dose vial **Physiodose** is hygienic, easy and safe to use.

INDICATIONS

Physiodose saline solution is recommended for babies, children, and adults:

- For nasal hygiene : to irrigate or cleanse dry or stuffed up noses,
- For ocular hygiene : as an eyewash or bath to cleanse the eyes,
- For auricular rinsing,
- For cleansing wounds.

COMPOSITION

Sodium chloride (NaCl) 0,9 g
Purified water q.s. 100 mL

PRESENTATION

Single-doses of 5 mL.

USE

Detach a vial and open it by twisting off the top.

FOR NASAL HYGIENE

To irrigate or cleanse dry or stuffed up noses, **Physiodose saline solution** can be used daily for nasal hygiene at a frequency of 1 to 6 washings per day, depending on the needs.

Instructions for use :

Tilt the head to the side. Delicately insert the tip into the upper nostril and gently squeeze the vial. Repeat the operation in the other nostril. Lift the head up to allow the mucus to run out and then wipe away the excess. Do not blow your nose violently for 10 minutes after administering.

Precautions for use :

For babies, squeeze drops with a minimum amount of pressure to avoid any risk of contamination of the middle ear.

FOR OCULAR HYGIENE

As an eyewash or bath to cleanse the eyes, **Physiodose saline solution** can be used daily for ocular hygiene at a frequency of 1 to 6 washings per day, depending on the needs.

Instructions for use :

Pour a few drops of solution into each eye, taking care not to touch the surface of the eye with the vial. Wipe away excess liquid with a gauze.

Precautions for use :

Wait 15 minutes before using any other eye treatment product.

FOR EAR HYGIENE

By rinsing. For perfect ear hygiene, use **Physiodose saline solution** to flush out solubilised earwax deposits and rinse the external auditory canal after using a surfactant auricular solution.

Instructions for use :

Rinse the external auditory canal with **Physiodose saline solution**. Use 1 vial of the product for both ears. Wipe away the surplus liquid with a gauze pad.

Warning :

Ear use of this medical device in cases of perforated eardrums, a history of ear surgery or chronic ear conditions is not recommended or should be advised in advance by your doctor or pharmacist.

FOR CLEANSING WOUNDS

Instructions for use :

Rinse the wound, using the entire contents of one vial and taking care not to touch the wound with the vial.

Precautions for use :

Wipe away the excess liquid from the unaffected area only.

WARNING

Do not use the same single-dose for the different uses. EXTERNAL USE. DO NOT INJECT. DO NOT SWALLOW. SINGLE USE.

Do not use a single dose that has already been opened or damaged because of the risk of rapid bacterial contamination.

If the symptoms persist or worsen, consult your doctor. Please return any major incident involving with this device to Laboratoires Gilbert - website: www.laboragilbert.fr and/or the national Competent Authority. KEEP OUT OF SIGHT AND REACH OF CHILDREN.

STORAGE CONDITIONS

Do not exceed the expiry date indicated on the single dose. Store at a temperature below 30°C.

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